



The Coag-Sense® PT/ INR System is the most Accurate, Precise, and Cost-Effective system in the marketplace.

Timing Is Everything In Anticoagulation Therapy!

When the right result is needed at the right time for medication management, choose the meter that doesn't use algorithms or estimates. The patient sample clotting time is the actual clotting time.



Achieves superior precision and accuracy with its micro-mechanical detection process

Produces a true prothrombin time—the only CLIA-Waived System of its kind

Displays PT results in less than one minute

Offers an INR range from 0.8 to 8.0

Requires no code chips

Reduces waste with individually packaged test strips—a more cost-effective system

How To Use The Coag-Sense® PT/INR Meter



› Step 1

Insert a test strip



› Step 2

Lance finger and collect a bead of blood with included transfer tube



› Step 3

Dispense sample onto test strip



› Step 4

Wait for results (testing time is the patient's actual clotting time)



Learn more at:

www.Coag-Sense.com/PT2Flyer



CoaguSense, Inc.
48377 Fremont Blvd., Ste. 113
Fremont, CA 94538 USA
www.coag-sense.com



Consult Instructions for Use